





REVOLUTION EVO

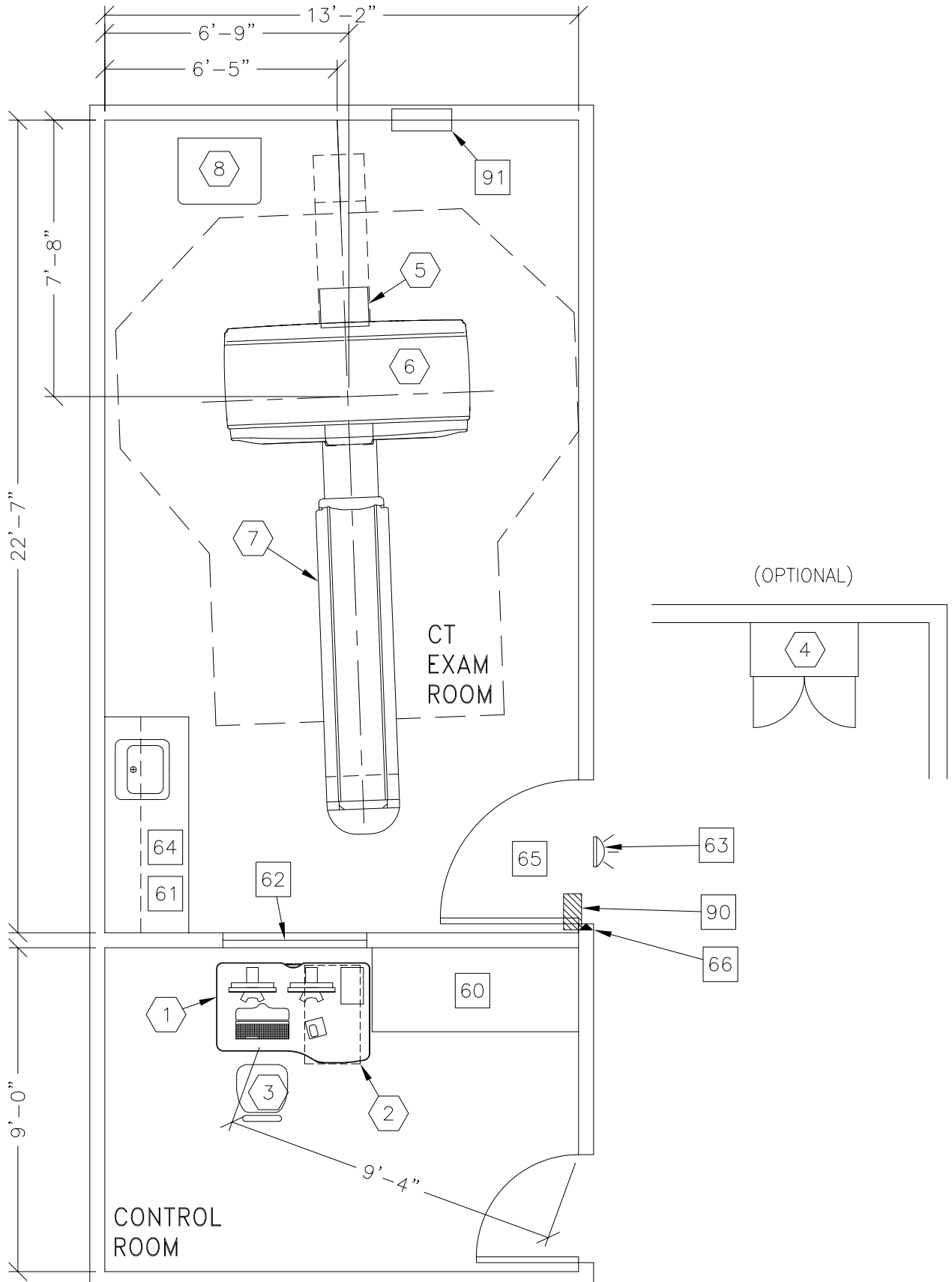

ROOM DIMENSIONS	LENGTH & WIDTH	CEILING HEIGHT
TYPICAL:	20'-8" x 13'-2" [6.30M x 4.00M]	9'-0" [2.74M]

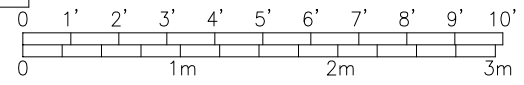

ROOM PLAN LOCATED ON PAGE 6-100

ITEM	QUANTITY	ITEM DESCRIPTION	WEIGHT	HEAT OUTPUT
①	1	OPTIMA DESK	125 lbs	
②	1	CONSOLE CABINET & LCD MONITORS	216 lbs	3201 btu
③	1	OPERATOR'S CHAIR		
④	1	STORAGE CABINET (EMPTY CABINET WEIGHT)	99 lbs	
⑤	1	REAR CABLE COVER		
⑥	1	REVOLUTION EVO GANTRY	4012 lbs	18703 btu
⑦	1	VT2000x PATIENT TABLE WITH EXTENDED TABLE TOP (W/675 lb PATIENT)	1796 lbs	1030 btu
⑧	1	POWER DISTRIBUTION UNIT OPTIONAL TABLE: GT2000 PATIENT TABLE WITH EXTENDED TABLE TOP (W/500 lb PATIENT)	815 lbs 1613 lbs	3412 btu 1030 btu
***** CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED ITEMS *****				
60		COUNTER TOP FOR EQUIPMENT-MINIMUM DEPTH 30 in. OR ADDITIONAL SHELVING MAY BE REQUIRED PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE INTERCONNECT CABLES TO RACEWAY BELOW COUNTERTOP.		
61		COUNTER TOP WITH SINK, BASE AND WALL CABINETS		
62		LEAD GLASS WINDOW		
63		X-RAY ON WARNING LIGHT - AVAILABLE FROM GE SUPPLY CALL: 800-200-9760 GE CAT. NO. WXIABWW-OF-XIU		
64		CASEWORK REQUIRED WITH A MINIMUM OF 16 CUBIC FEET TO STORE ALL SERVICE MATERIALS		
65		MINIMUM DOOR OPENING FOR EQUIPMENT DELIVERY IS 44 IN. W x 83 IN. H [1118mm x 2108mm], CONTINGENT ON A 96 IN. [2438mm] CORRIDOR WIDTH		
66		DOOR LIMIT SWITCH (REQUIRED IN SOUTH CAROLINA, OTHERWISE NEEDED ONLY IF REQUIRED BY STATE/LOCAL CODES)		
90		X-RAY ROOM WARNING LIGHT CONTROL PANEL REFERENCE JUNCTION POINT 'WLC' ON SHEET 'E1' FOR DETAILED DESCRIPTION -E4502RL FOR WARNING LIGHT CONTROL ONLY.		
91		MAIN DISCONNECT CONTROL GEMS CAT. NO. E4502AB 90 lbs., (IF A UPS IS IS NOT OR WILL NOT BE ORDERED, THE E4502AD CAN BE USED.)		

ENVIRONMENT

Ambient Operation Temperature: 64-79° (F) 18-26° C
 Allowable Temperature Change: 5° F [3° C] per hour
 Humidity: 30-60 percent non condensing
 Allowable Humidity Change: 5 percent per hour
 Max - Altitude: 7875 [2400M] ft. above sea level